THE NEW DIMENSION FOR THE LABORATORY

Hemostasis Analyzer

for clotting and chromogenic methods

Cat.No. 80 515 00

CE-approved

TEChrom IV plus

- Routine
- Emergency
- Research
- Stand-by
- Back-up

TEChrom IV plus mean new age in the Hemostasis

The **TEChrom IV plus** is a semi automated Hemostasis Analyzer, unique in flexibility to measure coagulation parameters (classical clotting tests and chromogenic substrates).

The **TEChrom IV plus** offers four different calculation models:

- Turbidity (clotting)
- Kinetic
- Endpoint
- 0,1 OD

Up to 16 different filters can be used (340, 405, 492, 550, 620 nm, etc.)

The **TEChrom IV plus** handle 30 different tests separate and individual with all specific calibration data.

The **TEChrom IV plus** fits perfectly into small and medium size Hemostasis laboratories and works excellent as a Stand-by unit to already installed automated instruments. For the usage in Emergency laboratories it is fully suitable.

TEChrom IV plus

Tests

PT, APTT, TT, FIB (Clauss/derived), Factor II, V, VII, VIII, IX, X, XI, XII, AT-III, Plasminogen, Heparin, C1-Esterase, Alpha 2-Antiplasmin, APC_R, Protein C, Protein S, PAI, tPA, μ PA, Heptest, D-Dimer, Lupus Anticoagulant, Factor XIII at 340 nm, PT-B (whole blood PT) at 710nm
Research test - free selectable parameters
STAT with 2xPT and 2xAPTT

Selectable dimensions:

s, %, Ratio, INR, Units (mg/dl), dE (Slope/min), E (Extinction) etc.

Development, Manufacturing and Sales:

Distribution through:

TECO Medical Instruments
Production + Trading GmbH
Dieselstrasse 1
D-84088 Neufahrn NB
Phone: +49-8773-910010
Fax.: +49-8773-910011
E-Mail: info@teco-gmbh.com
Internet: www.teco-gmbh.com

*Rev. B

Technical Description

The **TEChrom IV plus** is a micro-computer controlled measuring device for Hemostasis tests with a 4-channel optic and a LC-Display. An additional 4-times Stopwatch realizes time correct operating. The tests will be selected through keys on a foil keyboard. The dialog language is user friendly and selectable in English and German.

The Main program offers following menus

- ANALYSIS
- SETUP-TEST
- SETUP-SYSTEM
- SERVICE

The measurement can be started either manually by pressing the OPTIK-key in the same time when reagent was added, or with a connected Autopipette and electronic start. The result will be displayed in the specific dimension and can be converted and displayed in other dimensions by pressing the UNIT-key. With a connected Printer all results will be printed out automatically in all dimensions with date, Optic No. name of the test and the specific normal, therapeutical and critical ranges. Double determination with mean print-out is realised by duplication of the Patient-ID.

An additional Menu for SYSTEM ANALYSIS includes a variation of Service procedures, which increases the system reliability of the instrument and also from each measurement. Any errors in the system will be detected, displayed and printed out.

With the usage of the TECO Softwaretools the measuring curve data can be transmitted to a Personal Computer and loaded by any spread sheet with graphic function. Software updates are realized with an easy download from a computer to the instrument.

Technical Datas

Device

SMD Boards
V-25 Microcomputer
18-bit Analog-Digital Converter
1 MB Flash-EPROM
1 MB E² PROM

Power Supply

external Power Supply
Input voltage: 94 to 243 Vac
Input frequency: 50 to 60 Hz
Power output (max): 60 VA
Approvals:

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Optic block

4-channel Optics with Halogen lamp and intensity control
16 filters possible (coded)
Range: 0-2,5 OD linear to 3 OD max.
Resolution: 1 mOD

Keyboard

Foil keyboard, 4x8 matrix with Test-, Functions- and numerical keys

Display

4 lines x 20 Characters
Liquid Crystal Display

Incubation block

6 Cuvettes prewarming positions
1 Measuring position
2 Reagent positions Ø 22,5 mm
1 Reagent position Ø 32 mm, mixed
2 Reagent positions Ø 16 mm
Temperature: 37,0°C ± 0,2°C

Autopipette: (optional)

3-stage Autopipette with electronic start
Volume: 50/100/200 µl

Printer: (optional)

external, serial Thermal printer with Print-out of all results
Print-out of Test parameters
Print-out of System parameters
Service print-out with all specific System information

Standard package

1 Pc **TEChrom IV plus**
1 Pc Power Supply
1 Pc Mains cable
1 Pc Filter 405 nm
10 Pcs Cuvettes (4 Pos./ea)
10 Pcs Reagent container
4 Pcs Magnetic stirrer
1 Pc Reagent adaptor Ø 22,5mm
1 Pc Printer cable
1 Pc Pipette fastener
1 Pc Operation Manual
1 Pc Warranty card
1 Pc Installation report
1 Pc Service report

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